



DISPO COLOR®

Disposable Color Lens

DISPO COLOR® is an ideal daily wear, monthly disposable colored lens molded with a unique edging system. The Aspheric lens design provides outstanding visual performance. Each color of the DISPO COLOR® lens range is produced from a balance of three colors to create the spectacular change in the iris pattern, and allows your patients to have the fun of true colored lenses.

Lens Material

Methafilcon A material used in DISPO COLOR® is an ionic polymer with 55% water content. This material is easy to handle, maintains durability and possesses highly elastic properties that enhance the lens flexibility

Lens Specifications	
Material	Methafilcon A
Water Content	55%
Base Curve	8.70 mm
Diameter	14.40 mm
Center Thickness	0.07 mm@-3.00DS
Edge Thickness	0.07 mm
Oxygen Permeability (DK)	19.0×10^{-11} @ 35°
Oxygen Transmissibility (DK/L)	27.0×10^{-9} @ -3.00DS
Sphere Power	+4.00 to -6.00 in 0.25 DS steps
Lens Design	Aspheric front surface & spherical back surface
Color	Blue, Aqua, Green, Gray and Hazel
Packaging	2 Blisters per box



Fitting Instructions

1. Insert a clear DISPO® lens on the eye.
2. For cylinder powers between 0.50DS to -1.00DS add -0.25DS to the spherical power.
3. Allow lens to equilibrate for at least 20 minutes.
4. Evaluate lens fit. Lens should move about 0.5 mm each blink.
5. Evaluate acuity, perform over refraction if necessary, and adjust power.

Lens Care

- For ease and comfort, it is highly recommended to wet the lens for approximately 60 seconds before removing from the eye.
- Daily cleaning with all types of cold systems.